NL (Nederlands)

CAVEX BITE&WHITE ALIGNER WHITENING

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CAVEX BITE&WHITE ALIGNER WHITENING

Instructions for use

Cavex Bite&White Aligner Whitening is based upon HPS technology; a highly purified hydrogen peroxide in an aqueous gel with buffered neutral pH and fully stabilized to be stored at room temperature. The purified hydrogen peroxide breaks down rapidly into oxygen radicals and water when it comes in contact with the saliva present on the teeth. During the whitening process the free radicals attack the double carbon bonds that are the source of most colours (stains) in the tooth.

Content:

1x Cavex Teeth Whitening Pen 6 ml (ca 7.5 gr) pen.

Composition:

PEG-200, water, poloxamer 407, hydrogen peroxide, ammonium acryloyldimethylacrylate/VP copolymer, etidronic acid, mentha arvensis extract, potassium hydroxide, disodium EDTA, sodium stannate.

Indications for use:

Cavex Bite&White Aligner whitening is a tooth whitening gel used in combination with an aligner for use in lightening the colour of natural teeth. Cavex Bite&White Aligner whitening should only be dispensed by or on the order of a dental professional. Before any whitening procedure, a consultation with a qualified dentist is recommended to ensure underlying oral or health issues.

Warnings:

- Contains hydrogen peroxide.
- Avoid contact with eyes. Rinse immediately if product comes into contact with them.
- Keep out of reach of children.
- Avoid exposure of known sensitive teeth.
- Harmful when swallowed.

Hydrogen peroxide < 3%: is not to be used for persons under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

Storage advice:

- Store at a cool and dry place
- Avoid exposure to direct sunlight
- Store < 25 °C

Disposal:

Dispose of device in accordance with local waste regulations.

Instructions for use:

1. Prepare your teeth for the treatment by brushing them.
2. Remove the cap from the pen.
3. Turn the pen until a droplet of gel appears on the brush.
4. Apply a thick layer of gel directly onto the front surfaces of the teeth.
5. Close the pen with the cap after application.
6. Put the aligner in. Note: we recommend avoiding the treatment for the first two days of

a new aligner set to minimize the risk of sensitivity.

7. Wear the gel at least 20 minutes and maximum of 2 hours.

8. Rinse your mouth after treatment. Do not swallow rinsed gel.

9. Do not eat, drink or smoke tobacco for at least 30 minutes after each treatment.

10. Clean the aligner with a toothbrush and cold water after treatment.

First aid instructions:

- If a large quantity is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least 15 minutes and contact a doctor.
- If clothing, skin or hair contact occurs, remove contaminated clothing and rinse skin or hair with running water.

Potential adverse effects:

Adverse effects that may be associated with the use of Cavex Bite&White include, but are not limited to:

Sensitivity

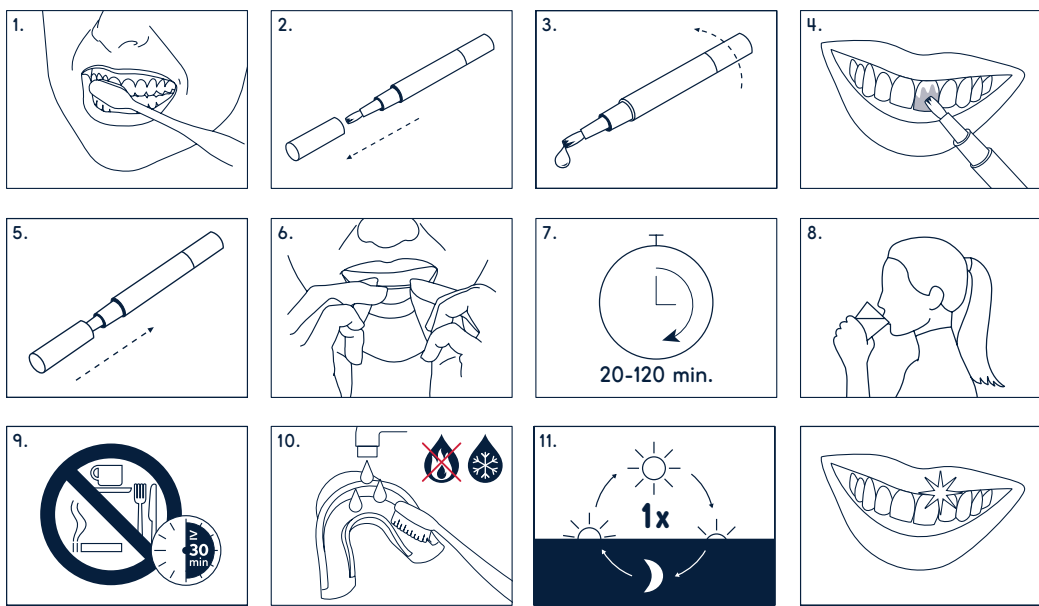
Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat. In general symptoms should disappear within several days after the treatment is stopped. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment.

Gingival irritation

Trays that extend onto the gingiva may cause mild to severe soft tissue irritation. If the trays fit it properly, check if the patient applies the proper quantity in the proper way. The patient may need to reduce the amount of gel in their trays.

Made by Cavex Holland BV

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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