# INSTRUCTIONS

CAVEX

CAVEX

# Bite & White

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## DESCRIPTION

Cavex Bite&White Ready 2 Use is based upon HPS technology; a highly purified hydrogen peroxide in an aqueous gel with buffered neutral pH and fully stabilized to be stored at room temperature. The purified hydrogen peroxide breaks down rapidly into oxygen radicals and water when it comes in contact with the saliva present on the teeth. During the whitening process the free radicals attack the double carbon bonds that are the source of most colours (stains) in the tooth.

## PACKAGING

- Cavex Bite&White Ready 2 Use, pre-filled bleaching trays in a vacuum packaging.

## COMPOSITION

Active ingredients:

- - 6 % Hydrogen peroxide

## INDICATIONS FOR USE

Cavex Bite&White Ready2Use is a whitening gel used to lighten the colour of natural teeth. Cavex Bite&White should only be dispensed by or on the order of a dental professional to people over the age of 18. A consultation with a qualified dentist is recommended before beginning the whitening procedure to check for any underlying oral or health issues.

## CONTRAINDICATIONS

It is recommended to consult a doctor, in any patient falling into one of the categories below is advised to consult a doctor prior to starting the treatment:

- Pregnant and lactating women
- Those currently treated for a serious illness

## CLINICAL PROCEDURE

- Record starting shade, review current dental health and make sure patient has realistic expectations about final results.
- Provisional or temporary restorations may discolour during the whitening process while composite or polycarbonate materials will remain unaffected. Therefore, patients with composite restorations must be advised that

existing composites may require replacement, since only natural teeth will whiten.

- For patients requiring aesthetic or adhesive restorations, wait 2 weeks after the last whitening treatment to allow for colour stabilization. All restorations should be intact or well sealed with adequate temporary materials.
- The use of a sensitivity relief product starting 2 weeks prior to the treatment procedure until 2 weeks after the treatment should be advised.
- Patients should always be warned of tooth sensitivity or gingival irritation during the treatment. The gingival areas that come in contact could turn red or white but will return to its normal colour after several days.
- To avoid soft tissue irritation, advise the patient to avoid overfilling trays, to wipe away any excess gel from the gum line and to rinse the mouth thoroughly after the treatment.
- The patient must not eat, drink, smoke or brush their teeth for at least 60 minutes after each treatment.
- Instruct the patient to stop the treatment and contact you immediately if any problem develops.
- The final shade will be achieved about 2 weeks after the start of the treatment. A follow-up appointment should be arranged after the treatment.

## WARNINGS

- Contains Hydrogen Peroxide.
- Avoid contact with eyes.
- Keep out of reach of children.
- Avoid exposure of known hypersensitive teeth.
- Do not eat, drink or smoke while wearing the filled custom-fitted trays.
- Harmful when swallowed.

## FIRST AID INSTRUCTIONS

- If a large quantity is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least 15 minutes and contact a doctor.
- If clothing, skin or hair contact occurs, remove contaminated clothing and flush skin or hair with running water.

## POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with the use of Cavex Bite&White include, but are not limited to:

### Sensitivity

Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat.

In general symptoms should disappear within several days after the treatment is stopped. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment.

### Gingival Irritation

Trays that extend onto the gingiva may cause mild to severe soft tissue irritation.

## STORAGE ADVICE

- Store at Room temperature (4 – 25°C)
- Do not use the material after the expiry date.

## DISPOSAL

- Dispose of device in accordance with local waste regulations

## Made in the Netherlands

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses.

The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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