-
PREMIUM WHITENING

CAVEX
BITE & WHITE
PATIENT KIT

-
Whitening system

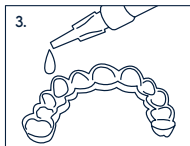
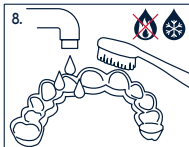
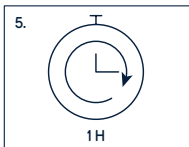
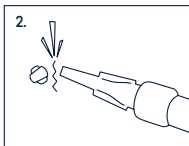
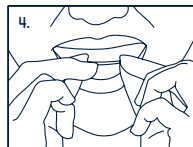
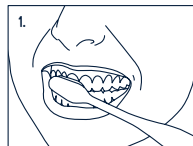
AT-HOME

INSTRUCTIONS

CAVEX

CAVEX BITE&WHITE PATIENT KIT

Instructions for use

1. Brush your teeth thoroughly.
2. Break the cap off the nozzle.
3. Administer 1 drop per tooth into the tray.

4. Insert the bleaching tray into your mouth.
5. Bleach for at least one hour.
6. Rinse your mouth with water.

7. Wait at least 30 minutes and then brush your teeth carefully.
8. Clean the bleaching tray with cold water and your toothbrush.

Hydrogen peroxide is the bleaching agent in Cavex Bite&White. It is added as carbamide peroxide in order to obtain a sufficiently stable product. The carbamide peroxide breaks down rapidly into oxygen radicals and water. During the whitening process the free radicals attack the double carbon bonds that are the source of most colours (stains) in the tooth.

EN

CONTENT

2x Cavex Bite&White 3 ml syringes

ACTIVE INGREDIENTS

16% carbamide peroxide
0.1% sodium fluoride

COMPOSITION

Glycerin, water, urea peroxide, polyacrylic acid, potassium hydroxide, potassium nitrate, sodium fluoride, EDTA, mentha arvensis extract.

INDICATIONS FOR USE

Cavex Bite&White is a whitening product used to lighten the colour of natural teeth. Cavex Bite&White should only be dispensed by or on the order of a dental professional to people over the age of 18. A consultation with a qualified dentist is recommended before beginning the whitening procedure to check for any underlying oral or health issues.

CONTRAINDICATIONS

It is recommended to consult a doctor, in case of:

- Pregnant and lactating women
- Those currently treated for a serious illness
- Children under the age of 18 years.

CLINICAL PROCEDURE

- Record starting shade, review current dental health and

make sure patient has realistic expectations about final results.

- Provisional or temporary restorations may discolour during the whitening process while composite or polycarbonate materials will remain unaffected. Therefore, patients with composite restorations must be advised that existing composites may require replacement, since only natural teeth will whiten.
- For patients requiring aesthetic or adhesive restorations, wait 2 weeks after the last whitening treatment to allow for colour stabilization. All restorations should be intact or well sealed with adequate temporary materials.
- The use of a sensitivity relief product starting 2 weeks prior to the treatment procedure until 2 weeks after the treatment should be advised.
- Patients should always be warned of tooth sensitivity or gingival irritation during the treatment. The gingival areas that come in contact could turn red or white but will return to its normal colour after several days.
- To avoid soft tissue irritation, advise the patient to avoid overfilling trays, to wipe away any excess gel from the gum line and to rinse the mouth thoroughly after the treatment.
- After each treatment, make sure patient replaces the syringe cap.

CAVEX BITE&WHITE PATIENT KIT

Instructions for use

- The patient should not eat, drink or smoke tobacco for at least 60 minutes after each treatment.
- Instruct the patient to stop the treatment and contact you immediately if any problem develops.
- The final shade will be achieved about 2 weeks after the start of the treatment.

WARNINGS

- Contains hydrogen peroxide.
- Contains sodium fluoride.
- Avoid contact with eyes. Rinse immediately if product comes into contact with them.
- Keep out of reach of children.
- Avoid exposure of known sensitive teeth.
- Do not eat, drink or smoke while wearing the filled custom-fitted trays.
- Harmful when swallowed.

Hydrogen peroxide < 6%; is not to be used for persons under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

FIRST AID INSTRUCTIONS

- If a large quantity (over 25% of the gel in the syringe) is swallowed, immediately drink water, do not induce vomiting and contact a doctor.
- If gel gets into eyes, hold eyelids apart and rinse the eye continuously with running water for at least 15 minutes and contact a doctor.
- If clothing, skin or hair contact occurs, remove contaminated clothing and rinse skin or hair with running water.

POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with the use of Cavex Bite&White include, but are not limited to:

Sensitivity

Some patients may experience increased tooth sensitivity to cold during the treatment, while others may have non-specific sensitivity in their teeth, gums, tongue, lips or throat. In general symptoms should disappear within several days after the treatment is stopped. The treatment can be proceeded with shorter, less frequent applications until the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment.

Gingival irritation

Trays that extend onto the gingiva may cause mild to severe soft tissue irritation. If the trays fit properly, check if the patient applies the proper quantity in the proper way. The patient may need to reduce the amount of gel in their trays.

STORAGE ADVICE

- Store refrigerated (<8°C)
- Store in a dark, dry place
- Avoid exposure to direct sunlight
- Do not use the material after the expiry date

DISPOSAL

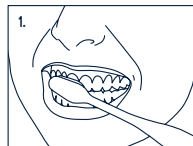
Dispose of device in accordance with local waste regulations.

Made by Cavex Holland BV

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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