
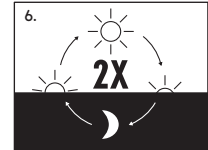
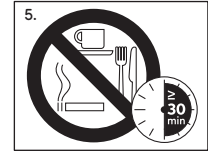
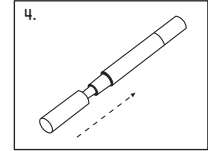
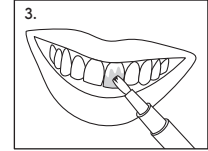
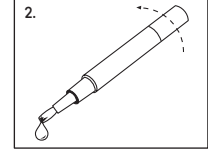
CAVEX BITE & WHITE TOOTH WHITENING PEN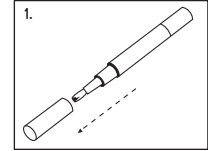

EN (English)

Cavex Bite&White - Tooth Whitening Pen

- **Content:** 1 x Cavex Tooth Whitening Pen 2 ml
- **Active ingredient:** ≤ 6 % Hydrogen peroxide
- **Composition:** PEG-200, water, hydrogen peroxide, poloxamer 124, t-butyl alcohol, etidronic acid, mentha arvensis extract, EDTA, sodium stannate.
- **Warning:** Contains hydrogen peroxide. Avoid contact with eyes. Rinse immediately if product comes in contact with eyes. Might cause sensitivity to teeth, gums, tongue, lips or throat. Hydrogen peroxide ≤ 6 % is not to be used for persons under 18 years of age; the product and treatment are only available from a dental practitioner and for the period stated.
- **Storage:** Store <25 °C.
- **Disposal:** Dispose of device in accordance with local waste regulations.

Instructions for use (numbers see images):

1. Remove the cap.
2. Turn the pen button until a small droplet of gel appears on the brush.
3. Apply a thick layer of gel directly onto the front surfaces of the teeth.
4. Replace the cap after application.
5. Do not eat, drink or smoke tobacco for at least 30 minutes after each treatment.
6. Apply the gel twice a day.

Made by Cavex Holland BV

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

■ Cavex Holland BV
Fustweg 5
2031 C.J Haarlem
The Netherlands
+31 23 530 77 00
dental@cavex.nl
cavex.nl

ZB708.02

DE (Deutsch)

Cavex Bite&White - Tooth Whitening Pen

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